Daisy

NASAL ASPIRATOR

AGU NS12

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TABLE OF CONTENTS

1	Introduction	39
2	List of symbols	40
3	Important information before use	41
4	Product overview.....	43
5	Battery installation / replacement	44
6	Children's tunes	45
7	Instruction for use	46
8	Care and maintenance	48
9	Troubleshooting	49
10	Technical specification	50
11	Technical data	51
12	Manufacturer's warranty.....	53

1 INTRODUCTION

Nasal aspirator **AGU NS12** is specially designed to clear mucus in newborn babies and infants in home–home environment and / or while travelling.

Music function

12 tunes in the memory of this device may interest and comfort a baby and help to make the clearing procedure more comfortable.

How to use

Insert aspiration tip into a nostril, after that the device is ready to remove nasal secretions.

Safety

The soft aspiration tip prevents the possibility for injury during operation.

Compact & Easy design

This device is compact and easy to hold and operate.

Washable parts

Silicone tips, collection cup & head module can be removed from the device and are easily washed. (Collection cup can also be disinfected in boiling water.)

Light weight, low noise

The child calmly responds to the procedure of cleansing the nose.

Battery

This device uses two 1.5 V AA batteries; it is convenient to replace the batteries anytime, anywhere.

Please read this User's manual to the end before you start using the nasal aspirator **AGU NS12**.

2

LIST OF SYMBOLS

Symbols Meaning

Warning / Note

The CE-mark of this device refers to the EC-directive 93 / 42 / EEC MDD

WEEE (Directive on Wasted Electrical and Electronic Equipment). This marking shown on the product indicates that it should not be disposed of with other household wastes at the end of its working life. To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this from other types of wastes and recycle it responsibly to promote the sustainable reuse of material resources.

Household users should contact either the retailer where they purchased this product, or their local government office, for details of where and how they can take this equipment for safe recycling.

Please read the instruction manual

Device classification type BF

Manufacturer

IP22

Meet BASIC SAFETY and ESSENTIAL PERFORMANCE after the IP22 conditioning test (IP22: Protection level against solid foreign objects & falling water).

The unit meets the main requirements of Customs Union Technical Regulation 020/2011 on Electromagnetic Compatibility of Technical Products. The unit meets the main requirements of Customs Union Technical Regulation 004/2011 «On the safety of low-voltage equipment», concerning the safety of low-voltage equipment.

3**IMPORTANT INFORMATION BEFORE USE**


The device is designed to remove nasal secretion and mucus in children (age 0~12 years old) in at-home environment.

1. **WARNING!** Do not modify this device in any way without authorization of the manufacturer.
2. Do not expose the device to high / low temperature, very high humidity level. Avoid exposing the device to direct sunlight.
3. Avoid extreme stress, shock or accidental dropping of the device.
4. Please replace all batteries when the power and/or the sound of the pump weakens during the use of the device.
5. It is recommended to remove the batteries if the device is not used for a long period of time.
6. Please press and hold the power button to start the device to generate vacuum for removing nasal secretion and mucus. If you want to stop the suction, simply release the power button to turn off the device.
7. The nasal aspirator is for cleaning / removing the nasal secretion only. Do not use this device other than the intended purposes.
8. During the use of the device, the babies should not hold their breath, close their mouth or swallow their saliva.
9. If the baby does not feel well when using the device, stop aspiration and consult a doctor immediately.
10. Do not apply excessive force when placing the aspiration tip into the nasal cavity.
11. This device is intended for use both at home and while traveling. Please consult your doctor before using the device in case of rhinitis and sinusitis and be aware of cleaning the aspiration tip before each use.
12. The intended operators for this device shall be parents or adults who can follow the instructions and notices in User's manual to operate the device. The device is not allowed to be used by children themselves. Keep the nasal aspirator stored away from the children. Do not let them operate the device themselves.
13. **DO NOT** use the nasal aspirator when children have the following contraindications:
 - nasal bleeding;
 - acute head, facial, or neck injury;
 - coagulopathy or bleeding disorder.


14. Before using the device, please read this User's manual carefully. Incorrect use of the device may cause the following hazards / complications:
 - mechanical trauma;
 - nasal bleeding;
 - damage of mucous membrane of the nose.
 15. Please keep the replaceable tips out of the reach of children children and pets cannot access to avoid risk of them to be swallowed.
 16. The tip may cause potential allergic reactions.
-  This nasal aspirator must only be used with AGU NS series tip. Please contact shops and pharmacies to buy those accessories.

4

PRODUCT OVERVIEW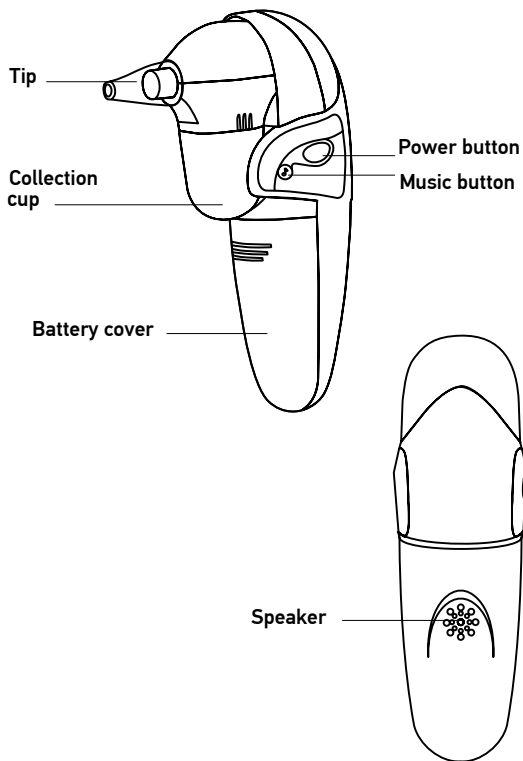

5

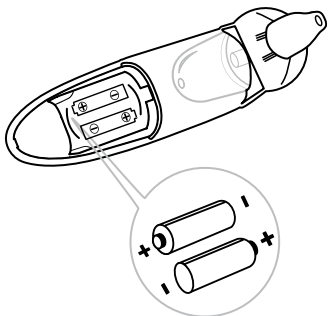BATTERY INSTALLATION/REPLACEMENT

1. Gently slide the battery cover back.
2. Carefully remove the old batteries and properly dispose them.
3. Insert or replace batteries into the battery compartment, ensuring to match the indicated polarity symbols. Always use new batteries.
4. Replace the battery cover.

NOTE


- Please do not expose the batteries to high temperatures and dispose them properly. Keep them out of children's reach.
- It is recommended to remove the batteries if the unit is not used for a long period of time.

Batteries and electronic devices should be disposed of in accordance with local recycling regulations and never dispose of them in the normal household waste.



6 CHILDREN'S TUNES

If you want to comfort your baby, just press the Music Button.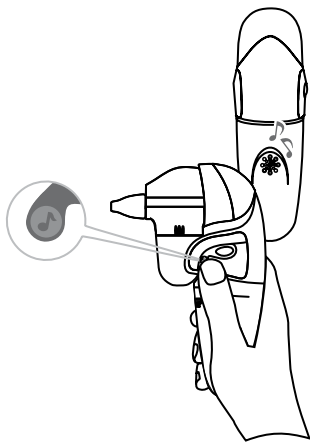


- Nasal aspirator **AGU NS12** provides 12 worldwide children's tunes.
- With the beautiful melodies, the baby will feel more comfortable while being treated with the device.
- To choose another melody, press the Music Button once again.
- To stop the melody, press and hold the Music Button.

7 INSTRUCTION FOR USE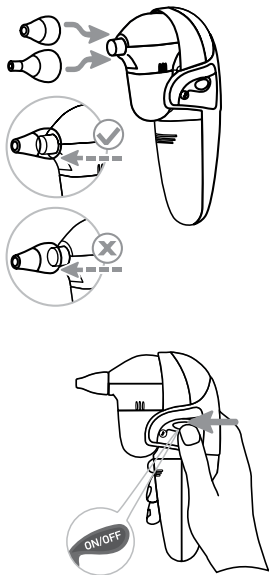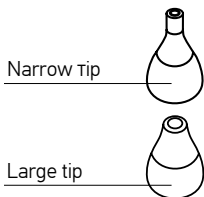

1. Choose the correct tip to use, according to shape and size of the nostrils of your baby.
 - The narrow tip is recommended for use when the mucus is loose and in a liquid form.
 - The large tip is recommended for use when the mucus is in a more solid form.
2. Be sure to place the tip securely on the aspirator. Make sure the tip is completely covering the rib before using.
 - ⚠ Check the tip before each use. If you notice any signs of wear or damage, change it immediately. The tips belong to consumables, their service life depends on the frequency of use of the device. We recommend replacing tips every six months of use.
3. Press and hold the power button while the device is clearing the mucus.

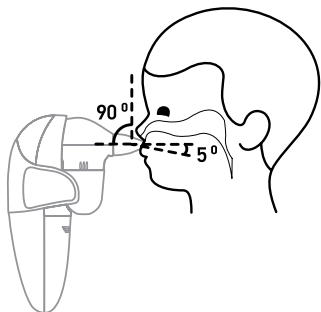4. Hold the baby upright and insert the tip into one nostril of the baby. Be careful not to press against the inside wall of the nostril or place the tip too far into the nostril. Be sure to hold the aspirator vertically keeping the tip of the aspirator at a 90° angle to the nasal passage. Mucus is cleared by the aspirator without any additional external force. Repeat the same procedure in the other nostril of the baby. If necessary, the procedure may be repeated.

! The mucus may not immediately appear in the collection cup. After each use it is recommended to do a quick cleaning of the tip by removing the tip from the aspirator.

5. Release power button to stop the suction.

! ATTENTION


In order to prevent getting liquid inside the main unit of the device and its failure:

- during the procedure, make sure that the amount of mucus in the storage tank does not exceed the maximum level (the maximum level is marked by a special mark on the tank);
- after completion of the aspiration procedure, immediately disconnect the head unit with the storage tank from the main unit.

If the device is used correctly, liquid will not get into the main unit. Please follow the recommendations above. Remember, getting liquid inside the device will lead to loss of warranty.

8 CARE AND MAINTENANCE


1. Separate the head module with the tip from the base of the aspirator by gently pulling it apart.
2. Unscrew collection cup by turning it clockwise.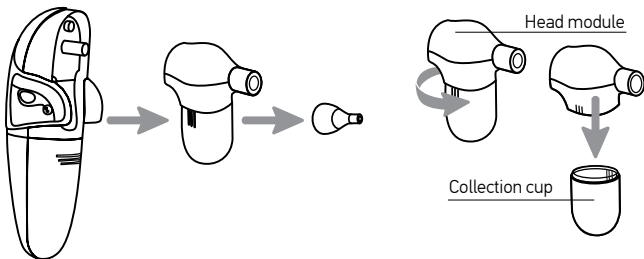

CLEANING PROCESS OF COLLECTION CUP, HEAD MODULE AND ASPIRATION TIP

It is recommended to wash the collection cup, head module and aspiration tip with mild soap in warm water for 3 minutes. Then rinse each part with running hot tap water for 3 minutes, disinfect them with boiling water. After the cleaning process, these parts must be completely dried after.


- ⚠ DO NOT** wash this device with any kinds of strong soap or cleaner. Please just wash the collection cup, head module, and tip with water and mild soap only.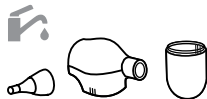

UNIT CLEANING

It is strongly recommended to wipe the unit with a clean soft dry cloth and keep the device clean.

- ⚠ DO NOT** wash the housing body of this device with water or any kind of liquids.

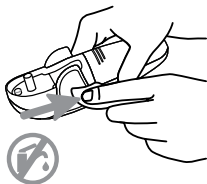9

TROUBLESHOOTING

If any trouble occurs while using the device, please check the following list first.

Problem	Possible Cause	Solution
The suction is weak or there is no suction at all	Batteries are low or improperly installed	Check or replace batteries
	Lack of grey rubber ring in the place where the collection cup is screwed in	Make sure there is a gray rubber ring in the place where the collection cup is screwed in
	The collection cup is not properly screwed in	Make sure the three guides on each side of the collection cup line up with the guides on the housing body of of the aspirator
The music will not play when the music button is pressed	Batteries are low or improperly installed	Check or replace batteries
The music will not stop when the music button is pressed	Pressing of the button is not long enough	To stop the music, keep pressing the button until the music stops playing

 **WARNING**

- If the device does not work even after taking the above-mentioned actions, please contact the store where it was purchased or the nearest dealer.
- Never turn the device on if any part of it is broken or does no longer work properly.

10

TECHNICAL SPECIFICATION

Product	Nasal aspirator AGU NS12
Model	AGU NS12
Strong power	effective vacuum pressure: 450 mmHg (60 kPa)
Sound pressure level	75 ~ 80 dB (30 cm away from the AGU NS12)
Aspiration tip	soft silicone
Motor	3V / DC
Operating conditions	temperature +16 °C~ +35 °C humidity up to 85 % (non-condensing)
Conditions for storage and transportation	temperature: -25 °C ~ +55 °C humidity: up to 85 % (non-condensing)
Power source	2 x 1.5 V AA alkaline batteries
Weight	146 g (without batteries)
Dimensions (L x W x H)	98,5 mm x 45,3 mm x 143 mm
Battery operation time	about 90 minutes (depends on the quality of batteries)
Typical service life	3 years
Completeness*	nasal aspirator, 1.5 V AA batteries (2 pieces), silicone tips (2 pieces), transparent collection cup, storage bag, user's manual, warranty coupon
Warranty period	2 years

***Notice:** the manufacturer has the right to introduce changes into the device design.

11 TECHNICAL DATA

GUIDANCE AND MANUFACTURER'S DECLARATION ELECTROMAGNETIC EMISSIONS

The **AGU NS12** nasal aspirator is intended for use in the electromagnetic environment specified below. The customer or the user of the **AGU NS12** nasal aspirator should assure that it is used in such an environment.

Emission test	Compliance	Electromagnetic environment guidance
RF emissions CISPR 11	Group 1	The AGU NS12 nasal aspirator uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment
RF emissions CISPR 11	Class B	The AGU NS12 nasal aspirator is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not applicable	

GUIDANCE AND MANUFACTURER'S DECLARATION ELECTROMAGNETIC IMMUNITY

The **AGU NS12** nasal aspirator is intended for use in the electromagnetic environment specified below.

The customer or the user of the **AGU NS12** nasal aspirator should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment guidance
Electrostatic discharge (ESD) IEC 61000-4-2	+ 6 kV contact + 8 kV air	+ 6 kV contact + 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %
Power frequency (50 / 60 Hz) magnetic field IEC 61000-4-8	3 A / m	3 A / m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment

<p>Radiated RF IEC 61000-4-3</p>	<p>3 V / m 80 MHz to 2,5 GHz</p>	<p>3 V / m</p>	<p>Portable and mobile RF communications equipment should be used no closer to any part of the AGU NS12 nasal aspirator, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance $d = 1,2 \sqrt{P}$ $d = 1,2 \sqrt{P}$ 80 MHz to 800 MHz $d = 2,3 \sqrt{P}$ 800 MHz to 2,5 GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey ^a, should be less than the compliance level in each frequency range ^b.</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> 
--------------------------------------	--	----------------	---

NOTE1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a. Field strengths from fixed transmitters, such as base stations for radio (cellular / cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the **AGU NS12** nasal aspirator is used exceeds the applicable RF compliance level above, the **AGU NS12** nasal aspirator should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the **AGU NS12** nasal aspirator.

b. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V / m.

RECOMMENDED SEPARATION DISTANCES between portable and mobile RF communications equipment and the AGU NS12

The **AGU NS12** nasal aspirator is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the **AGU NS12** nasal aspirator can help prevent electromagnetic interference by maintaining a minimum distance

between portable and mobile RF communications equipment (transmitters) and the **AGU NS12** nasal aspirator as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter, W	Separation distance according to frequency of transmitter (m)		
	150 kHz to 80 MHz $d = 1,2 \sqrt{P}$	80 MHz to 800 MHz $d = 1,2 \sqrt{P}$	800 MHz to 2,5 GHz $d = 2,3 \sqrt{P}$
0,01	0,12	0,12	0,23
0,1	0,38	0,38	0,73
1	1,2	1,2	2,3
10	3,8	3,8	7,3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where p is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

12 MANUFACTURER'S WARRANTY

A warranty for defects in material and assembly cover the device for a period of 2 years from the date of purchase. The warranty does not apply to battery. Carefully follow the instructions to ensure reliable and long-term operation of the device.

If the device does not work properly, contact the authorized seller.

ATTENTION

Do not attempt to repair the nebulizer by yourself; it will lead to a loss of warranty.

Changes made to the nebulizer without the manufacturer's permission will lead to a loss of warranty.

For repair and maintenance, please contact an authorized after-sales service.

The manufacturer reserves the right to change and amend the design of the device.

*The addresses of the service centers are specified on the website agu-baby.com.

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AViTA Corporation 9F, No.78, Sec.1, Kwang-Fu Rd.,
San-Chung District, 24158 New Taipei City, Taiwan (R.O.C.)
No.858, Jiao Tong Road, Wujiang Economic Development Zone
Jiangsu Province, P.R.C. Postcode: 215200 Made in P.R.C.

Authorized Representative in the EU: Wellkang Ltd.,
Suite B, 29 Harley Street, London W1G 9QR,
England, United Kingdom.

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Make sure that children could not use the appliance without supervision, because some small parts can get into the respiratory tract or the digestive system.

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RoHS
COMPLIANT

AGU[®] ADVANCED
GROWING
UP

AGU[®] is the registered trademark by Montex Swiss AG,
Tramstrasse 16, CH-9442, Berneck, Switzerland