Penguin

ASPIRATOR

NASAL

AGU NS19

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EN USER MANUAL

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TABLE OF CONTENTS

1	Introduction	35
2	List of symbols	36
3	Important information before use	37
4	Product overview.....	39
5	Battery installation/replacement.....	40
6	Children's tunes	41
7	Instruction for use	42
8	Care and maintenance	44
9	Troubleshooting	45
10	Technical specification	46
11	Conditions of storage, transportation and operation.....	47
12	Disposal.....	47
13	Certification.....	48
14	Manufacturer's warranty.....	48

1 INTRODUCTION

Nasal aspirator **AGU NS19** is specially designed to clear mucus in newborn babies and infants in home environment and/or while travelling.

Music function

12 tunes in the memory of this device may interest and comfort a baby and help to make the clearing procedure more comfortable.

How to use

Insert aspiration Tip into a nostril, after that the device is ready to remove nasal secretions.

Safety

The soft aspiration Tip prevents the possibility for injury during operation.

Compact & easy design

This device is compact and easy to hold and operate.

Washable parts

Silicone Tips, Collection cup & head module can be removed from the device and are easily washed (Collection cup can also be disinfected in boiling water).

Light weight, low noise

The child calmly responds to the procedure of cleansing the nose.

Battery

This device uses two 1.5 V AA batteries; it is convenient to replace the batteries anytime, anywhere.

Please read this User's manual to the end before you start using the nasal aspirator **AGU NS19**.

2

LIST OF SYMBOLS

Symbols Meaning

Warning/Note.

- The product meets the basic requirements of:
- Directive 2014/30/EU EMC on electromagnetic compatibility;
 - Directive 2011/65/EU RoHS on the restriction of the use of certain hazardous substances in electrical and electronic equipment.

WEEE (EC Directive on Waste Electrical and Electronic Equipment).

This marking means that the device should not be disposed of together with household waste. In order to prevent a possible damage to the environment and harm to human health, such waste should be segregated from other types of waste and disposed of in accordance with the relevant accepted standards.

Please read the instruction manual.

Manufacturer.



3**IMPORTANT INFORMATION BEFORE USE**

The device is designed to remove nasal secretion and mucus in children (age 0~12 years old) in at-home environment.

1. **WARNING!** Do not modify this device in any way without authorization of the manufacturer.
2. Do not expose the device to high / low temperature, very high humidity level. Avoid exposing the device to direct sunlight.
3. Avoid extreme stress, shock or accidental dropping of the device.
4. Please replace all batteries when the power and/or the sound of the pump weakens during the use of the device.
5. It is recommended to remove the batteries if the device is not used for a long period of time.
6. Please press and hold the Power button to start the device to generate vacuum for removing nasal secretion and mucus. If you want to stop the suction, simply release the Power button to turn off the device.
7. The nasal aspirator is for cleaning / removing the nasal secretion only. Do not use this device other than the intended purposes.
8. During the use of the device, the babies should not hold their breath, close their mouth or swallow their saliva.
9. If the baby does not feel well when using the device, stop aspiration and consult a doctor immediately.
10. Do not apply excessive force when placing the aspiration Tip into the nasal cavity.
11. This device is intended for use both at home and while traveling. Please consult your doctor before using the device in case of rhinitis and sinusitis and be aware of cleaning the aspiration Tip before each use.
12. The intended operators for this device shall be parents or adults who can follow the instructions and notices in User's manual to operate the device. The device is not allowed to be used by children themselves. Keep the nasal aspirator stored away from the children. Do not let them operate the device themselves.
13. **DO NOT** use the nasal aspirator when children have the following contraindications:
 - nasal bleeding;
 - acute head, facial, or neck injury;
 - coagulopathy or bleeding disorder.


14. Before using the device, please read this User's manual carefully. Incorrect use of the device may cause the following hazards/complications:
 - mechanical trauma;
 - nasal bleeding;
 - damage of mucous membrane of the nose.
 15. Please keep the replaceable Tips out of the reach of children and pets cannot access to avoid risk of them to be swallowed.
 16. The Tip may cause potential allergic reactions.
-  This nasal aspirator must only be used with AGU NS series Tip. Please contact shops and pharmacies to buy those accessories.

4 PRODUCT OVERVIEW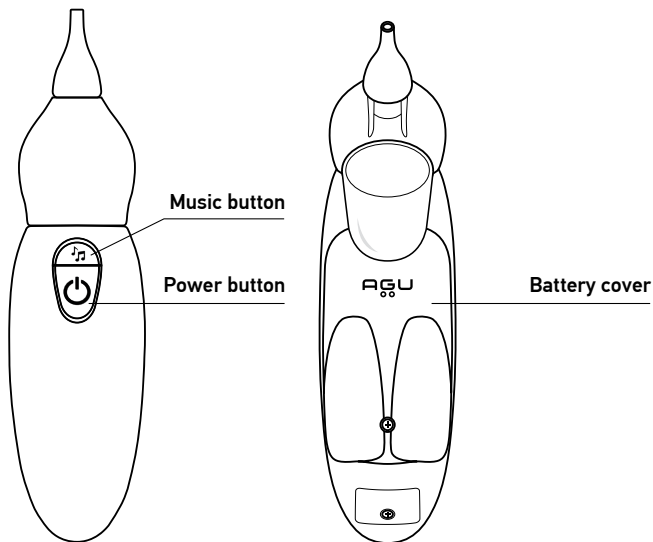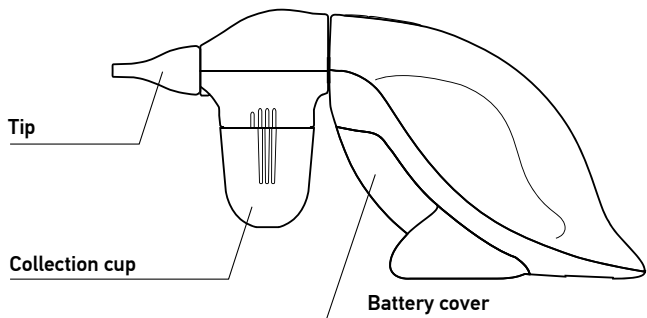

5

BATTERY INSTALLATION/REPLACEMENT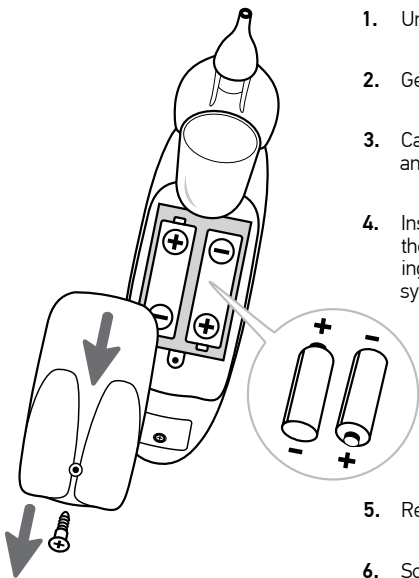

1. Unscrew the screw.
2. Gently slide the Battery cover back.
3. Carefully remove the old batteries and properly dispose them.
4. Insert or replace batteries into the battery compartment, ensuring to match the indicated polarity symbols. Always use new batteries.
5. Replace the Battery cover.
6. Screw in the screw.


⚠ NOTE

- Please do not expose the batteries to high temperatures and dispose them properly. Keep them out of children's reach.
- It is recommended to remove the batteries if the unit is not used for a long period of time.

 Batteries and electronic devices should be disposed of in accordance with local recycling regulations and never dispose of them in the normal household waste.

6 CHILDREN'S TUNES

If you want to comfort your baby, just press the Music button.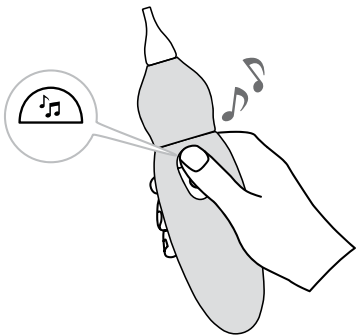

- Nasal aspirator **AGU NS19** provides 12 worldwide children's tunes.
- With the beautiful melodies, the baby will feel more comfortable while being treated with the device.
- To choose another melody, press the Music button once again.
- To stop the melody, press and hold the Music button.

7 INSTRUCTION FOR USE

Narrow Tip


Large Tip

1. Choose the correct Tip to use, according to shape and size of the nostrils of your baby.



- The narrow Tip is recommended for use when the mucus is loose and in a liquid form.
- The large Tip is recommended for use when the mucus is in a more solid form.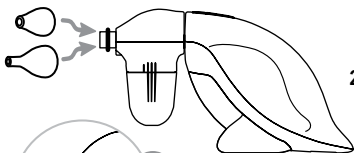


2. Be sure to place the Tip securely on the aspirator. Make sure the Tip is completely covering the rib before using.

Check the Tip before each use. If you notice any signs of wear or damage, change it immediately. The Tips belong to consumables, their service life depends on the frequency of use of the device. We recommend replacing Tips every six months of use.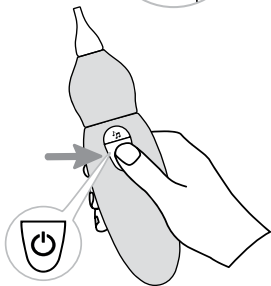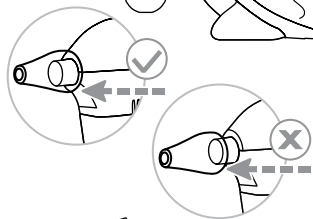

3. Press and hold the Power button while the device is clearing the mucus.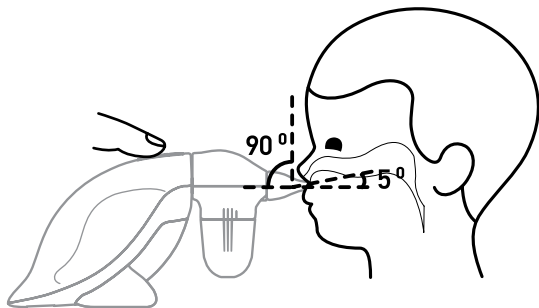

4. Hold the baby upright and insert the Tip into one nostril of the baby. Be careful not to press against the inside wall of the nostril or place the Tip too far into the nostril. Be sure to hold the aspirator horizontally keeping the Tip of the aspirator at a 90° angle to the nasal passage. Mucus is cleared by the aspirator without any additional external force. Repeat the same procedure in the other nostril of the baby. If necessary, the procedure may be repeated.

⚠ The mucus may not immediately appear in the Collection cup. After each use it is recommended to do a quick cleaning of the Tip by removing the Tip from the aspirator.

5. Release Power button to stop the suction.

⚠ ATTENTION


In order to prevent getting liquid inside the main unit of the device and its failure:

- during the procedure, make sure that the amount of mucus in the storage tank does not exceed the maximum level (the maximum level is marked by a special mark on the tank);
- after completion of the aspiration procedure, immediately disconnect the head unit with the storage tank from the main unit.

If the device is used correctly, liquid will not get into the main unit. Please follow the recommendations above. Remember, getting liquid inside the device will lead to loss of warranty.

8 CARE AND MAINTENANCE

1. Separate the head module with the Tip from the base of the aspirator by gently pulling it apart.
2. Unscrew Collection cup by turning it clockwise.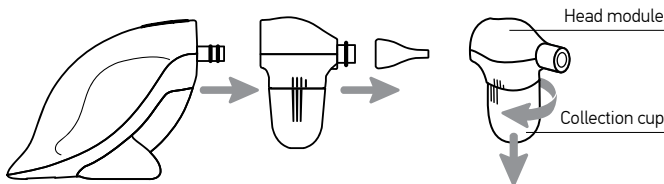

CLEANING PROCESS OF COLLECTION CUP, HEAD MODULE AND ASPIRATION TIP

It is recommended to wash the Collection cup, head module and aspiration Tip with mild soap in warm water for 3 minutes. Then rinse each part with running hot tap water for 3 minutes. After the cleaning process, these parts must be completely dried after.


⚠ DO NOT wash this device with any kinds of strong soap or cleaner. Please just wash the Collection cup, head module, and Tip with water and mild soap only.

⚠ DO NOT expose the device details to high temperatures (more than 60°C).

UNIT CLEANING

It is strongly recommended to wipe the unit with a clean soft dry cloth and keep the device clean.

⚠ DO NOT wash the housing body of this device with water or any kind of liquids.

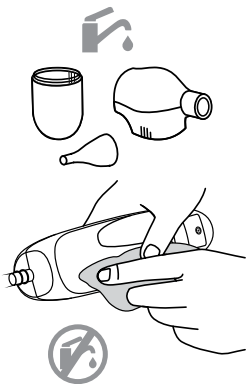9

TROUBLESHOOTING

If any trouble occurs while using the device, please check the following list first.

Problem	Possible Cause	Solution
The suction is weak or there is no suction at all	Batteries are low or improperly installed	Check or replace batteries
	Lack of grey rubber ring in the place where the Collection cup is screwed in	Make sure there is a gray rubber ring in the place where the Collection cup is screwed in
	The Collection cup is not properly screwed in	Make sure the three guides on each side of the Collection cup line up with the guides on the housing body of the aspirator
The music will not play when the Music button is pressed	Batteries are low or improperly installed	Check or replace batteries
The music will not stop when the Music button is pressed	Pressing of the button is not long enough	To stop the music, keep pressing the button until the music stops playing

WARNING

- If the device does not work even after taking the above-mentioned actions, please contact the store where it was purchased or the nearest dealer.
- Never turn the device on if any part of it is broken or does no longer work properly.

10

TECHNICAL SPECIFICATION

Product	Nasal aspirator AGU NS19
Model	AGU NS19
Maximum discharge pressure	450 mm Hg (60 kPa)
Sound pressure level	55-60 dB (30 cm away from the AGU NS19)
Aspiration Tip	soft silicone
Motor	3V/DC
Operating conditions	temperature +16 °C ~ +35 °C humidity up to 85% (non-condensing)
Conditions for storage and transportation	temperature: -25 °C ~ +55 °C humidity: up to 85% (non-condensing)
Power source	2 x 1.5 V AA alkaline batteries
Weight	150 g (without batteries)
Dimensions (L x W x H)	170 mm x 43 mm x 86 mm
Battery operation time	about 90 minutes (depends on the quality of batteries)
Typical service life	3 years
Completeness*	nasal aspirator, 1.5 V AA batteries (2 pieces), silicone tips (2 pieces), transparent collection cup, storage bag, user's manual, warranty coupon
Warranty period	2 years

***Notice:** the manufacturer has the right to introduce changes into the device design.

11 CONDITIONS OF STORAGE, TRANSPORTATION AND OPERATION

- The device should be transported at -10°C to $+40^{\circ}\text{C}$ with a maximum relative humidity of 75 % or less.
- The device can be operated at a temperature of $+10^{\circ}\text{C}$ to $+40^{\circ}\text{C}$.
- Multifunctional electric kettle AGU NS19 should be stored at a temperature between $+10^{\circ}\text{C}$ and $+40^{\circ}\text{C}$ with a maximum relative air humidity of not more than 65%.
- Do not subject the unit to sudden temperature fluctuations.

ATTENTION!

After transport or storage at low temperatures, it is necessary to keep the device at room temperature for at least 2 hours before switching on.

12 DISPOSAL

The symbol on the product or on its packaging indicates that this product is not classified as household waste.

- Disposing of this device in a proper way you will help to prevent possible adverse consequences for the environment and people's health
- For environmental protection purposes, this device should not be disposed of together with domestic (household) waste. The disposal should be performed in accordance with local regulations and laws.
- The device should be disposed of in accordance with the EU Directive 2012/19/EU WEEE (Waste Electrical and Electronic Equipment) for waste electrical and electronic equipment.

If you have any questions, please contact the local utility responsible for waste disposal.

13 CERTIFICATION

Nasal aspirator **AGU NS19** complies with:

- the main requirements of the EU Directive 2014/30/EU on electromagnetic compatibility, and Directive 2011/65/EU on the restriction of the use of certain hazardous substances in electrical and electronic equipment;
- the main requirements of TP TC 004/2011 «On the safety of low-voltage equipment», concerning the safety of low-voltage equipment.

14 MANUFACTURER'S WARRANTY

The warranty period of nasal aspirator AGU NS19 is 24 months from the date of sale.

The warranty is valid subject to availability of a warranty card filled out by a duly authorized representative confirming the date of sale and also of the relevant receipt. The warranty does not cover the consumable parts, components and constituent elements, namely: batteries, silicone narrow and large tips, transparent collection cup, storage bag.

The warranty and free maintenance service are not provided in case:

- the device is used with violation of the requirements set by the operation manual;
- damage is caused by consumer's willful or wrong actions as a result of improper or negligent use;
- traces of mechanical effect (dents, cracks, chips, etc.), tampering, dismantling, traces of repairs outside the authorized technical service center, traces of ingress of moisture into the case or effect of corrosives, or of any other outside interference with the device design are observed on the device case, and also in other cases of violation by the consumer of the rules of storage, cleaning, transportation and maintenance of the device stipulated by the operation manual;
- penetration of oil, dust, insects, liquids (not intended for the use with devices) and other foreign matters inside the device.

The warranty does not cover the defects (faults) of the device caused by the following reasons:

- normal wear and tear of component parts with a limited service life;
- damage caused to accessories and nozzles which do not form an integral part of the device silicone narrow and large tips , transparent collection cup, storage bag, package;
- deposition of sediment on the medication chamber (grid / membrane) irrespective of the used inhalation solution;
- use of low-quality accessories and nozzles, or the accessories and nozzles which reached the end of their service life;
- force majeure circumstances (accident, fire, flood, electrical power failures, etc.).

WARNING

Strictly follow the instructions in order to ensure reliable and long-term operation of the device.

Apply to the sales assistance in case the device does not work properly.

For the repair and maintenance apply to a specialized service department.

The manufacturer reserves a right to change the design of the device.

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AGU[®] is the registered trademark by Montex Swiss AG,